

Healthcare Particulate Respirator and Surgical Mask FN-N95-2020H PLUS



Description & Classification

Model	FN-N95-2020H PLUS
Product Name	Healthcare Particulate Respirator and Surgical Mask
Style	Cup
Strap Type	Headstrap
Size	Standard, Fits most adults
Sample	
Measurements	W: 5.2" x H: 4.9"
Colour	White
Manufactured In	Canada
Use Range	Medical Facilities Long term Care Facilities Personal Protection

Material and Performance Specification

Material	Non-woven Fabric
Contains Latex	No
Particle Filter Efficiency	≥96.6%
Inhalation Resistance	≤12.57
Exhalation Resistance	≤12.53
Fluid Resistance	120mm Hg
Flammability Rating	Class I

Performance Requirements

Test	Health Canada Requirements	NIOSH Requirements	FN-N95-2020H PLUS
Particulate Filter Efficiency	≥95%	≥95%	Pass
Inhalation Airflow Resistance	< 35 mmH ₂ O (343.2 Pa)	< 35 mmH ₂ O (343.2 Pa)	Pass
Exhalation Airflow Resistance	< 25 mmH ₂ O (245.2 Pa)	< 25 mmH ₂ O (245.2 Pa)	Pass
Mechanical Strength of Head Strap	10N (0.98kg) Per Headstrap	2.2kg Per Strap	Pass
Fluid Resistance	ASTM Level 1 for 80 mm Hg ASTM Level 2 for 120 mm Hg ASTM Level 3 for 160 mm Hg	ASTM Level 1 for 80 mm Hg ASTM Level 2 for 120 mm Hg ASTM Level 3 for 160 mm Hg	Pass, Level 2
Flammability Rating	Class 1	Class 1	Pass, Class 1



Packaging Specification	
Respirator Weight	0.32 oz
Respirators in Box	20
Weight of Box with Respirators	0.54 lbs
Box Dimensions	L: 5.6" x W: 5" x H: 7.87"
Respirators Per Case	180
Boxes Per Case	9
Weight of Case	6 lbs
Case Dimensions	L: 20" x W: 15.5" x H: 8.75"
Respirators Per Pallet	7560
Cases Per Pallet	42
Pallet Dimensions	L: 41" x W: 48" x H: 66"

INSTRUCTIONS FOR USE

Donning Procedure



1. Hold the respirator in your hand, with the nosepiece at your fingertips, allowing the head straps to hang freely under your hand.
2. Place the respirator to your face, covering your chin and nose. Using your opposite hand, pull the top strap over your head, positioning it at the top of your head. Pull the bottom strap and position it below the ears.
3. Adjust the nose strip to make the respirator fit snugly around the nose and cheeks.
4. Make sure there is nothing between the respirator and your face. Check that the mask is tight so that it fits properly, and that no air is leaking. If air is leaking, follow the steps to readjust the respirator.

Removal Procedure



Without touching the respirator, slowly lift the bottom strap from below your ears and over your head. Then, lift off the top strap. Discard used product in accordance with applicable regulations.

WARNING



HEALTHCARE PARTICULATE RESPIRATOR AND SURGICAL MASK

For proper respirator use user must be trained and comply with the instructions on the box. For assistance see your supervisor or call 1(866) 791-0909.

WARNING
AVERTISSEMENT

MASQUE FILTRANT À PARTICULAIRES DE SOINS MÉDICAUX ET MASQUE CHIRURGICAL

Pour l'utilisation correcte du respirateur l'utilisateur doit être qualifié et respecter les consignes sur la boîte. Pour de l'aide voir le superviseur ou bien appeler 1(866) 791-0909.